In gomma • Bilamellata
Made of rubber • Two-side ribbed

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EN | HOT WATER BOTTLE - Made of rubber · Two-side ribbed

Instructions: Hold the neck of the bottle upright and slowly fill it to 2/3 of its capacity with hot water not exceeding 50°C (**Fig.1**). Carefully let the air out of the bottle by lowering it onto a flat surface until the water reaches the opening (**Fig.2**). Close the opening of the bottle with the cap supplied, taking care to screw it in as far as it will go (**Fig.3**). Check that the cap is properly screwed shut and that there are no leaks.

Warning: Do not use water at a temperature above 50°C. Do not microwave. The hot water bottle can cause scalding. Avoid prolonged and direct contact with the skin. It is recommended to always use the hot water bottle with a cover or cloth. Do not fill the hot water bottle directly from the tap, as this can considerably shorten its service life. Keep out of reach of children. Do not use as a cushion. Do not sit on the bottle. Avoid contact with oil or grease. Store away from light and heat sources. Check both bottle and cap for signs of wear and damage before use. Keep these instructions for future reference.

Duration: The hot water bottle is subject to natural wear and tear. For proper storage, it is recommended not to place anything on top of the bottle when it is stored. When not in use, empty the bottle completely and store it, without the cap, in a cool, dry and dark place. If the bottle is damaged, replace it. Use within 2 years of the date on the packaging.

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SISMA S.p.A.